

Spraying instructions

Please read the following spraying instructions in conjunction with user guidance prior to using Granulox® for the first time.

1. 360° Spraying

Sprays evenly in all directions from every position, especially in hard to reach areas.

2. There is no need to shake

Granulox® remains at a constant consistency, so there is no need to shake. Unlike conventional spray cans, shaking gives no indication of the amount of Granulox still remaining. On average, a can lasts for 30 treatments. Empties 99% of contents.

3. On first use, nonstop spraying may occur


Nonstop spraying may occur on first time use, due to the very tight construction of the nozzle. If so, simply jiggle the nozzle gently back and forth and guide it slightly upwards. Spraying will then stop.

When using Granulox for the first time, you may choose to spray into a tissue or over a sink.

4. Nozzle may clog


Due to the very fine valve, you may find that the nozzle can clog. If this happens, then switch to a new nozzle. A pack of new nozzles is included in every box.

- Simply remove the old nozzle by tugging smartly and twisting simultaneously.
- To insert a new nozzle, push down while twisting.
- If nonstop spraying occurs, please follow procedure as described in the point 3 above.

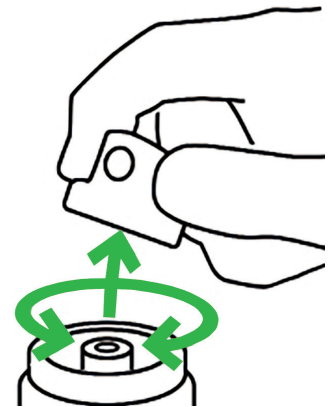

Advanced technology for guaranteed sterility

Granulox® uses an innovation in non-aerosol spray cans. Content remains free of contamination during the entire life of the product.

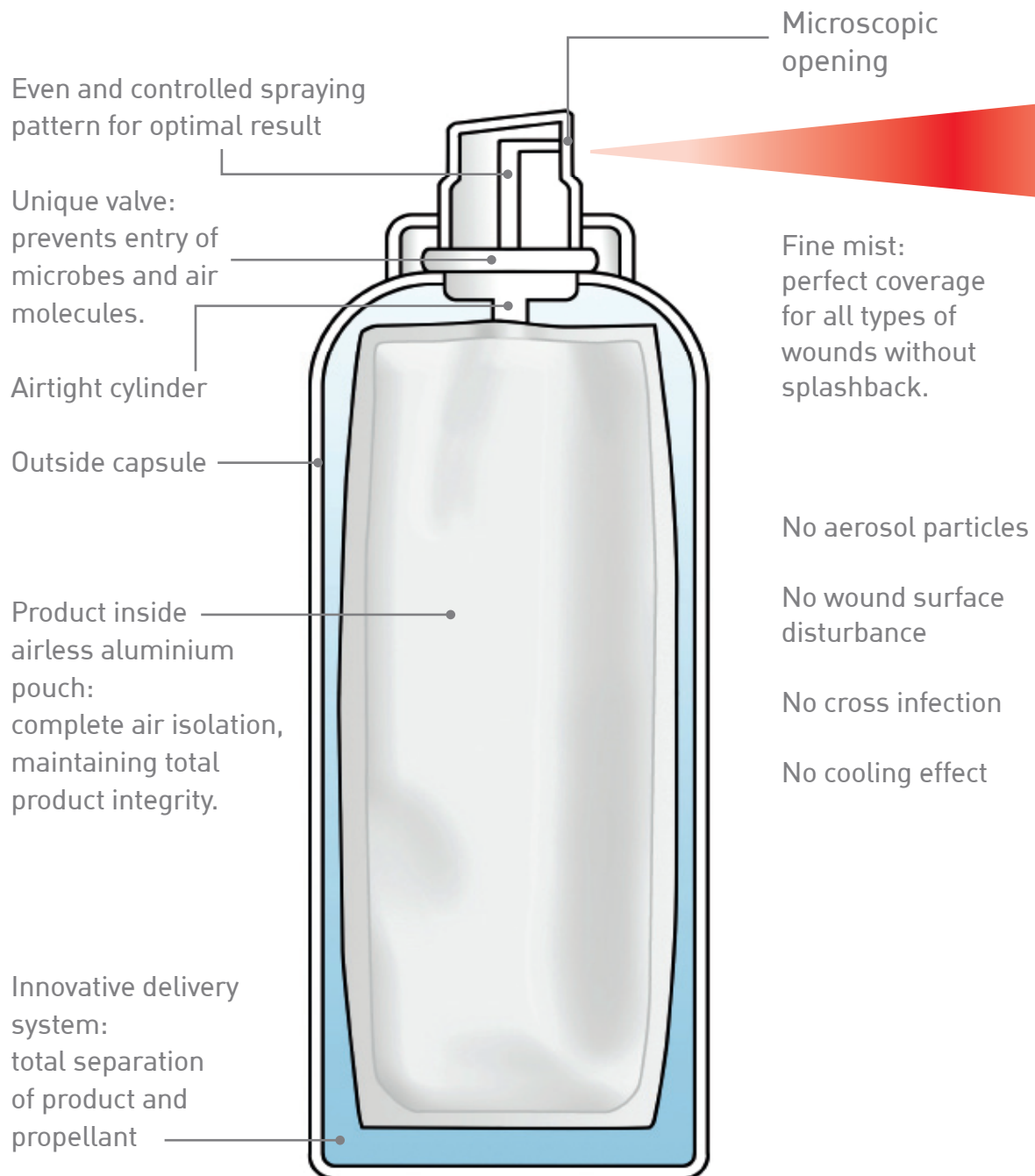

Find out more at www.molnlycke.com

Mölnlycke Health Care AB, Box 13080, Gamlestadsvägen 3C, SE-402 52 Göteborg, Sweden. Phone + 46 31 722 30 00.
The Mölnlycke and Granulox trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies. © 2019 Mölnlycke Health Care. All rights reserved. HQIM000907


Mölnlycke®